

# Non-Surgical Isolation Gowns

(T Gown - Open Back)

SOLUTIONS FOR MEDICAL PROVIDERS AND FIRST RESPONDERS



These polyethylene, non-surgical isolation gowns are easy to put on (don) and take off (doff). The full length, long sleeve style provides full coverage. The sleeves include a thumb loop to secure the sleeve.

Under AAMI Guidelines, protective apparel must meet minimum performance standards for strength, barrier protection and fluid resistance to ensure the safety of healthcare workers. These gowns have been tested for compliance with ANSI/AAMI PB70:2012 Minimal Barrier Performance Level 1, Level 2 requirements.

## FEATURES

- Compliant with ANSI/AAMI PB70:2012 Minimal Barrier Performance Level 1, Level 2 requirements
- Easy to put on and take off
- Openings in end of sleeve seal for fingers/hand
- Long sleeve, full coverage
- Made from polyethylene (Virgin LLDPE)
- Non-sterile

(Donning/Doffing instructions on reverse)

### WARNING

- This product is **NOT** sterilized.
- This product is for **single use ONLY**.
- Do **NOT** use in the presence of any high intensity heat source or flammable gas.



SKU	Model	Neck	Mil	Film Color	Material	Style	Span	Length	Case Pack	Cases/Pallet
TGOWNLS2	74 - L	12	1.5	Blue	Polyethylene	Individually Folded	74"	50"	100	40
TGOWNXLS2	76 - XL	12	1.5	Blue	Polyethylene	Individually Folded	76"	54"	100	32
1018256	74 - L	12	1.5	Blue	Polyethylene	Individually Folded	74"	50"	100	40
1018310	76 - XL	12	1.5	Blue	Polyethylene	Individually Folded	76"	54"	100	32

Manufactured in USA / Patent Pending

For more information contact [ppeproducts@novolex.com](mailto:ppeproducts@novolex.com)

# Non-Surgical Isolation Gowns

## DONNING AND DOFFING INSTRUCTIONS



1. Locate the print on the back side of the gown.



2. Unfold the gown so that the print is facing you.



3. Open the gown and put your arms through the sleeves.



4. At the end of each sleeve there are open perforations for your thumb and hand.



5. Secure the gown.



Option 1: Overlap the sides and secure with enclosed tape as shown.



Option 2: Tie gown at the neck and then wrap the extra body material to the back side and tie as shown.

6. Wrap the sleeve at the wrist and tuck corner into palm.



7. Put on gloves according to CDC guidelines.



**Do off the gown per CDC guidelines.**

